

# walk with independence



Model #4010

**Cruiser De-Light**  
Folding Walker



**ORTHO-MED, INC.**  
*Commitment to Caring*  
NovaOrtho-Med.com

## **USER WARNING**

Read Immediately

**DO NOT USE AS A WHEELCHAIR OR TRANSPORT CHAIR.**  
Use seat for stationary seating only. Never sit and push with your feet. Do not go up or down stairs or use on escalators.



Fig. 1

### **To Sit Down:**

---

1. Lock the brakes.
2. Hold onto both handgrips, not the brake handles.
3. Turn around and touch the back of your legs against the seat.
4. Slowly lower yourself onto the seat.

### **To Prevent Tipping Over when Sitting:**

Distribute your weight evenly on the seat.

Do not lean on one side of the walker.

Both feet should rest flat on the ground.

## Usage Instructions


**Before Use:** Adjust all four legs to desired height. Make sure all four legs are set to the same height and the walker is fully level on an even surface.

### To Walk

1. Place seat in down position so plastic seat arms rest on walker frame
2. Place hands on grips and lightly grip the brake handle. (Do not squeeze handle when walking)


### To Park and Sit

1. Be sure the seat is in the downward position
2. Push down on the ends of the handles until they lock in the downward position.
3. Test the walker before sitting down to make sure it will not roll.
4. Before sitting, touch the back of your legs to the edge of the seat to ensure proper positioning of the walker.

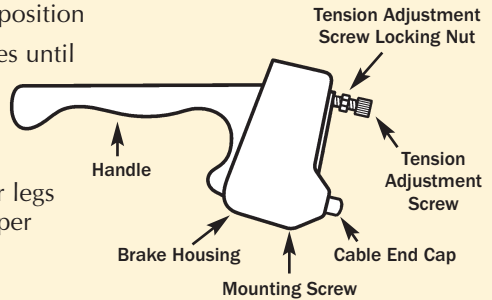

### To Fold

1. Lift seat up so it rests in the upward position
2. Pull release pins on each side and fold walker legs inward.

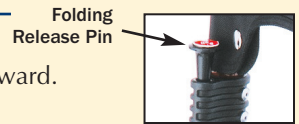

### How to use the Hand Brakes

1. To lock - Push down on the ends of the handles until they lock in the downward position.
2. To release – Squeeze handle until handle releases from the downward position

### How to Adjust the Brakes

The following information is designed to assist you with the adjustment of your Nova Feather Touch Hand Brake System.

#### 1. Hand brake is too loose.

Loosen the tension adjustment screw locking nut and turn the screw one full turn clockwise (as viewed from standing at the back of the walker). If this does not correct the adjustment, repeat one full turn at a time until you have adjusted your brakes. Then make sure the locking nut is spun tight against the housing to prevent the screw from loosening.

#### 2. Hand brake is too tight.

Loosen the tension adjustment screw locking nut and turn the screw one full turn counter clockwise (as viewed from standing at the back of the walker). If this does not correct the adjustment, repeat one full turn at a time until you have adjusted your brakes. Then make sure the locking nut is spun tight against the housing to prevent the screw from loosening.

## Customer Registration & Warranty

1. Do not attempt to repair the walker yourself.
2. Call the store where you purchased the walker. Most dealers do repairs and will re-order parts for you.
3. If your dealer is unable to help you, please call Nova at (310) 352-3600.

### Lifetime Limited Warranty

NOVA Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product.

NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products.

For warranty service, please contact the dealer from whom you purchased your NOVA O.M. product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate the nature of the defect, the product serial number, and the name and location of your dealer. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

This warrant does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, hand grips, casters, and other parts subject to normal "wear and tear."

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Record this information for future reference:

Serial # \_\_\_\_\_

Date of purchase \_\_\_\_\_

Purchased from \_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

(Store name, address, phone)



**ORTHO-MED, INC.**  
*Commitment to Caring*

**800 557-6682**

**NovaOrtho-Med.com**