# LIFETIME LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**12 HARBOR PARK DRIVE  
PORT WASHINGTON, NY 11050  
MADE IN CHINA**

© 2002 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA

## **Premium Series Bath Bench Banco para Baño Serie Premium Banc de bain Série Premium**

For use with item# 12500

**NOTE:** Before assembly, insure that all parts and quantities match parts list. Inspect all components for shipping damage. If any parts are determined to be missing or damaged, contact your dealer/retailer for replacements.

## WARNING!

DO NOT use this equipment without first reading and understanding its instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer, or technical person before installing. Otherwise, injury or damage may occur. These shower seats are not designed to support the total weight of an individual. Use arms for assistance only.

## PART LIST

1. Seat – 1 each
2. Leg – 4 each
3. Arms – 2 each\*
4. Back – 1 each \*\*


\*Arms are sold separately under item# 12503.

\*\* Back available with item# 12502

## ASSEMBLY INSTRUCTIONS

1. Place seat (1) upside down on a flat surface. Insert leg (2) into the corner recess of the seat by applying a firm downward pressure until leg is fully engaged. Repeat step #1 for all 4 legs.
2. For model with arms (3) place chair upright and insert arms into holes in side of seat until arm is fully engaged.
3. For models with a back place back in the slot, which is in the rear of the seat, and insert back until engaged.

To adjust legs for proper height or, uneven surfaces, rotate the leg extension and disengage from leg. Then insert leg extension into desired slot until adjustment is fully engaged.

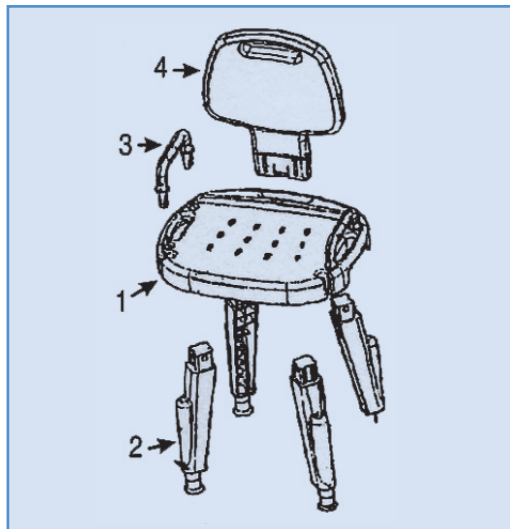## DISASSEMBLY INSTRUCTIONS

1. The back has a lock tab that, when depressed, will allow the back to be pulled out of their slots freely.
2. To remove arms, pinch together the tabs on the arm which, when assembled, will be under the seat.
3. It is not recommended to remove legs as this may cause damage to the products. The legs are not designed to be removed after assembly.

## CARE AND MAINTENANCE

1. Do not use an abrasive cleaner. Use a non-abrasive cleaner with warm water.
2. Periodically inspect all components for damage or looseness.

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