

LIFETIME LIMITED WARRANTY

Your product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Cane Instructions & Warranty



SAFETY SUMMARY

To ensure the safe operation of DRIVE CANES, these instructions MUST be followed.

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment-otherwise, injury may occur.

Each individual should always consult with their physician or therapist to determine proper adjustments and usage.

Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.

Canes can provide ambulatory assistance for an individual weighing up to 300 pounds. The canes are designed to provide support, increased stability and assistance to the end use while walking. The canes are NOT designed to support the total weight of individuals.

ADJUSTABLE CANE/QUAD CANE WARNINGS

Single button MUST be fully engaged and protruding through height adjustment hole on cane handle to ensure a positive lock. ALWAYS test to see that cane handle is properly and securely locked in place BEFORE using. When using a Quad Cane, the cane should be placed on the walking surface with all four legs in contact with, and vertical tube perpendicular to, the walking surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury. Extra care should be exercised when walking on a slippery or wet surface.

SAFETY SUMMARY

ADJUSTABLE CANE/QUAD CANE WARNINGS

If a cane is exposed to extreme temperature (above 100° F or below 32° F), high humidity and/or becomes wet, prior to use, ensure handgrips do not twist on cane handle-otherwise damage may occur.

For stability, ensure that longest/or flattest portion of base is closest to foot-otherwise, injury or damage may occur. It may be necessary to adjust Quad Cane for right-hand or left-hand use.

FOLDING CANE/ORTHOPEDIC GRIP CANE WARNINGS

Ensure that the snap button fully protrudes through adjustment hole of cane. This ensures that the cane is securely locked in position.

BEFORE using the Folding Cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.

PROPER CANE HEIGHT (FIGURE 1)

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.

FOLDING CANES, ADJUSTABLE CANES, ORTHOPEDIC GRIP CANES AND QUAD CANES – Adjust cane so top portion of handle touches inside of wrist where it meets palm.

FIGURE 1



ADJUSTING CANES(FIGURE 2)CANE HEIGHT

1. perform one (1) of the following:

FOLDING CANE, ORTHOPEDIC GRIP CANE, ADJUSTABLE CANE, OFFSET CANE – Turn the anti-rattle collar **COUNTERCLOCKWISE** to loosen.

QUAD CANE – Loosen lock knob by turning **COUNTERCLOCKWISE**.

2. Press IN snap button.
3. Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.

NOTE: AN AUDIBLE “CLICK” WILL BE HEARD WHEN SNAP BUTTON SECURELY PROTRUDES THROUGH THE ADJUSTMENT HOLE.

4. Repeat STEPS 3-4 until cane is at height determined **PROPER CANE HEIGHT** in this instruction sheet.
5. perform one (1) of the following:

FOLDING CANE, ORTHOPEDIC GRIP CANE, ADJUSTABLE CANE, OFFSET CANE – Turn the anti-rattle collar **CLOCKWISE** to tighten.

QUAD CANE - Tighten lock knob by turning **CLOCKWISE**.

ADJUSTING QUAD CANES FOR RIGHT OR LEFT HAND USE

1. Loosen lock knob
2. Depress the snap button and turn cane handle 180 degrees until stet button locks into same height adjustment hole on other side of cane.
3. When cane handle is firmly positioned at desired height and snap button protrudes through height adjustment hole on cane handle, turn lock knob clockwise to secure.

CARE AND MAINTENANCE

1. Ensure snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.

Replace any broken, damaged or worn items immediately.

FIGURE 2

