



Quantity:
One Pair

ALUMINUM CRUTCHES

Push-Button/Adjustable
Fully Assembled

- 10400-8
Adult
Patient Height
5'2" to 5'10"



- 10401-8
Youth
Patient Height
4'6" to 5'2"



- 10402-8
Tall
Patient Height
5'10" to 6'6"



CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Keep Hand Grip Wing Nut tightened.
- Use extra caution on wet and slippery surfaces.

THIS PRODUCT CONTAINS DRY NATURAL RUBBER


Made in CHINA

ALUMINUM CRUTCHES

Push-Button/Adjustable
Fully Assembled

For maximum fit
and assembly, see
instructions below:

1. Height: Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push-buttons snap back into place.
2. Handgrip: Adjust handgrip to fit an approximate 10" bend at patient elbow.
3. Safety Check: With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and handgrip nuts are tightened securely.

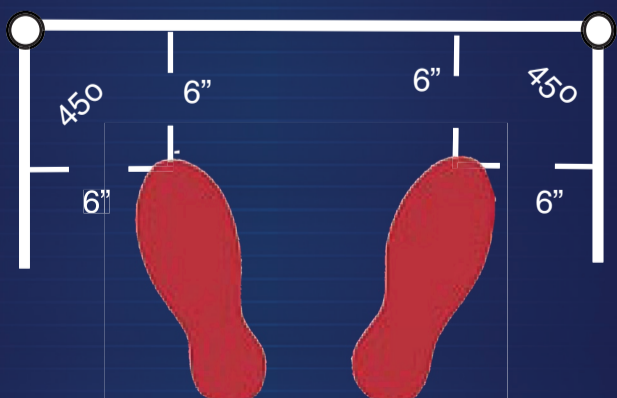

CRUTCH FITTING POSITION

THIS PRODUCT CONTAINS DRY NATURAL RUBBER