www.drivemedical.com

Item # 11145-1

Folding Commode

Silla con Orinal - Plegable

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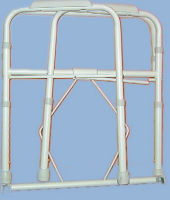
Instructions & Warranty
Instrucciones y Garantía
Instructions et Garantie


www.drivemedical.com

Commode Assembly Instructions


1.

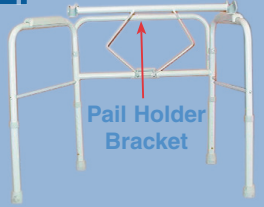
2.

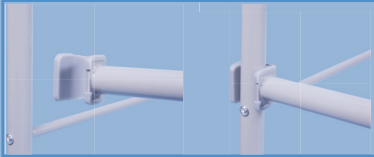3.

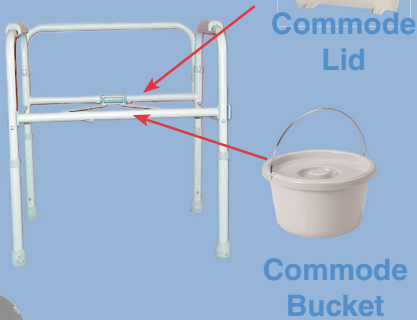4.

7.

www.drivemedical.com

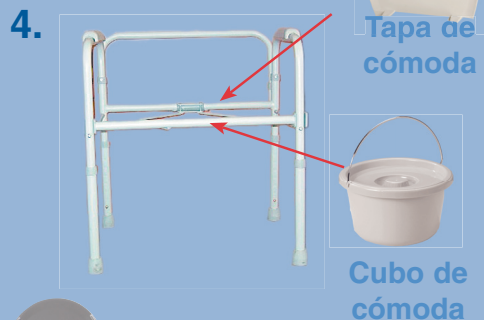
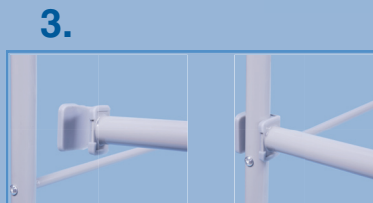
Assembling Commode:

1. Remove packing material from commode parts.
2. Unfold commode and stand up.
3. Adjust the two sides of the commode so the pail frame can be lowered onto the side frames until locked securely.
4. Place splash shield onto commode frame. Place bucket into splash shield.
5. Attach seat and lid by pressing down on rear of commode frame.
6. Attach back support by holding both ends while depressing brass push pins. Place back support into frame receptacle and push down until the brass push pins "snap" through frame holes.
7. Attach thumb screw in front to secure commode frame to side frame.

Disassembling Commode:

1. Remove lid, seat, bucket and splash guard.
2. Remove back by depressing push pins and pulling up on back support.
3. Un-attach thumb screw.
4. Pull up on front of commode frame until the guides are clear from side frames.
5. Push side frames away from commode frame to allow the commode frame to fold down.
6. Swing one side of side frame to back of commode.
7. Swing the other side of the cross frame to the front of commode.

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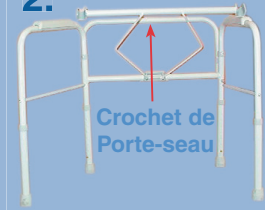
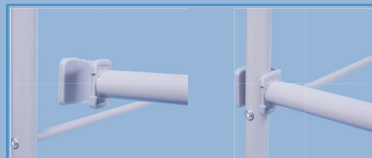
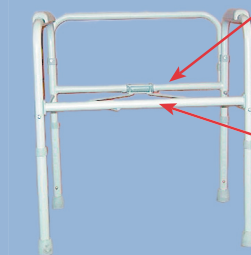
LIFETIME LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

Port Washington, NY 11050

MADE IN CHINA

© 2002 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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©2002 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2002 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.