

LIFETIME LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



Port Washington, NY 11050

MADE IN CHINA

© 2002 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China



BATH BENCH Instructions & Warranty

www.drivemedical.com

Assembly Instructions

(Read Instructions Carefully
Before Installing)



1 Position tub and shower seat leg assembly so that center flattened areas overlap, as shown in Figure 1.

2 Place plastic tub and shower seat upside down on a work surface and position leg assembly so that the 5 holes in the leg assembly are aligned with the pre-threaded sockets in the bottom of the tub and shower seat. Insert smaller 1-1/2" bolts through holes with lock nut and curved washer, and tighten with screwdriver, as shown in Figure 1. **DO NOT OVERTIGHTEN AS IT COULD STRIP THE THREADS.**

3 If you are assembling shower seat with back, follow same instructions, except place back assembly over leg assembly before inserting larger 2-1/2" bolts provided, as shown in Figure 2.

WARNING: Use only the stainless steel hardware provided to avoid decolorization and dangerous weakening of the hardware from rust or corrosion.

Periodically check assembly for continued tightness. NEVER USE BATH SAFETY SEAT IF ANY LOOSENESS IN THE ASSEMBLY IS NOTICED, AS PERSONAL INJURY MAY RESULT. SIT. DO NOT STAND on this seat to avoid a possible dangerous fall.

Meets ANSI Standard A117.1 Specifications.
300 lb Weight Capacity.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Tools Needed for Installation: Phillips Head Screwdriver.

