



Item # 14029

Operating Instructions:

1. Place the pump on a flat surface or suspend pump on end of bed with attached hook.
2. Place the mattress on top of bed mattress.
Note foot end.
3. Connect air hoses to pump and mattress.
4. Check and ensure the air hoses are not kinked or tucked under mattress.
5. Turn pressure to firm. Plug the power cord into outlet. Switch on the pump. The green switch will light when unit is operating.
6. Adjust pressure for patient comfort

Danger:

DO NOT use in the presence of flammable anesthetic:
It may cause possible explosive hazard.

Caution:

Risk electric shock. DO NOT remove cover (or back).
Refer to your qualified service personal for assistance.

Warranty Information

Your Drive brand product is warranted to be free of defects in materials and workmanship for 1 year of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.